

Aqua-Sonic Cleaner

#4045



800.423.3294
www.westernoptical.com



**Thank you for selecting the Western Optical Supply, Inc.
AQUA-SONIC Ultrasonic Cleaner #4045**

The AQUA-SONIC cleans in a very unique way. An energy pulse compresses then releases each water molecule creating a scrubbing action. An energy pulse is rated at 42,000Hz and is microseconds long. Each second the water molecules contract and expand thousands of times. The AQUA-SONIC is high tech, high performance, in a compact design.

Precautions:

NEVER operate The AQUA-SONIC when dry.

Do not operate without spectacles in the bath to absorb the energy.

Do not place your hand into the bath while the unit is activated.

Cleaning the stainless steel tank with chlorine-based cleanser or chlorinated water, will eat through the stainless steel pan.

Instructions:

Plug The AQUA-SONIC into a 120 volt outlet (draws 35 watts). Before turning on fill the stainless steel tank with water and cleaning fluid to within 3/4" of the top. Place spectacles to be cleaned into the tank with the lenses facing up before activating. Close the lid and depress the ON button to activate the three-minute timer.

For best performance always use Western's Biodegradable Ultrasonic Cleaning Solution #4042 according to the directions on the container. Ultrasonic cleaning fluid assists the energy pulses to scrub out grime. Replace the water and cleaning solution often to avoid bacterial growth and to keep The AQUA-SONIC operating at peak performance.